

PHG Protective
Health Gear

Particulate Respirator N95

MADE IN U.S.A.
of globally sourced
materials

National Institute for
Occupational Safety and Health
NIOSH[®]



National Institute for Occupational Safety and Health
Approval: TC-84A-PH15
P/N 879 N95


Distributed by:
Safety Emporium
PO Box 1003
Blackwood, NJ 08012
Ph: (866) 326-5412 toll-free
Fax: (856) 553-6154
esupport@safetyemporium.com
www.safetyemporium.com

Assigned to: Protective Health Gear LLC
35-57 Kentucky Ave.
Paterson NJ 07503
Tel: +1 (800) 225 - 1025

Filtering Facepiece Respirator
for Public Health Emergency Protection:
Read and understand UI prior to use.
User must adhere to all C&L in the UI.

The following Cautions and Limitations spelled out in the UI apply:
A, B, C, J, L, M, N, O, P, PH

Before occupational use of this respirator, a written respiratory protection program must be implemented meeting all the local government requirements. In the United States, employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing, and applicable OSHA substance specific standards.

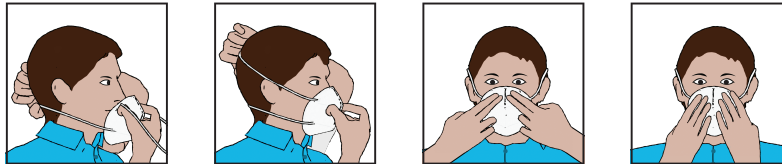

- 1 Unfold the respirator.
- 2 Place the respirator against your face with the noseclip across the bridge of your nose. Hold the respirator on your face and position the respirator under the chin.
- 3 Pull the bottom strap over your head, position it around the neck, below the ear.
- 4 Pull the top strap over your head resting it high at the top of your head.
- 5 Place the fingertips of both hands at the top of the metal nosepiece. Mold the nosepiece to the shape of the nose bridge by pushing inwards while moving your fingertips down both sides of the nosepiece.
- 6 The respirator must be checked before each use. To perform the user seal check, place both hands completely over the respirator, being careful not to disturb the position, and exhale sharply. If air leaks around your nose, adjust the nose clip as described in Step 5. If air leaks at respirator edges, adjust the straps back along the sides of your head. Perform seal check again if an adjustment is made. If you cannot achieve a proper fit, see your supervisor. Do not enter an area requiring respirator use.

MADE IN U.S.A.
of globally sourced
materials

Model 5160

Protective Health Gear LLC
35-57 Kentucky Ave.
Paterson NJ 07503
Tel: +1(800)225-1025
Fax: 973-689-6163
Sales@protectivehealthgear.com
www.protectivehealthgear.com



Particulate Respirator N95

Model
5160

PHG Protective
Health Gear

Particulate Respirator N95

Respirateur N95 contre les particules

Respirador contra partículas N95

Distributed by:
Safety Emporium
PO Box 1003
Blackwood, NJ 08012
Ph: (866) 326-5412 toll-free
Fax: (856) 553-6154
esupport@safetyemporium.com
www.safetyemporium.com


Elastic headstraps for secure fit

Fitted nose clip for comfort

4 layer mask

Premium filter

latex free

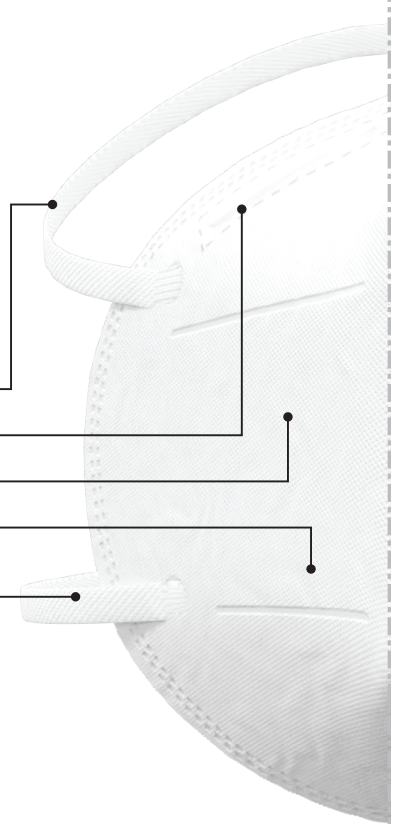

50 Pieces Per Carton
Individually Packed



MADE IN U.S.A.
of globally sourced
materials



Particulate Respirator N95

Protections

N95-Particulate Filter (95% filter efficiency level) Effective against particulate aerosols free of oil; time use restrictions may apply.

Cautions and Limitations

- A** Not for use in atmospheres containing less than 19.5 percent oxygen.
- B** Not for use in atmospheres immediately dangerous to life or health.
- C** Do not exceed maximum use concentrations established by regulatory standards.
- J** Failure to properly use and maintain this product could result in injury or death.
- L** Follow the manufacturer's User Instructions for changing cartridges, canister and/or filters.
- M** All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
- N** Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O** Refer to User Instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- P** NIOSH does not evaluate respirators for use as surgical masks.
- PH** The limited FFR approval will be effective until the declaration that circumstances exist justifying the authorization of the emergency use of personal respiratory protective devices during the COVID-19 outbreak is terminated under Section 564(b)(2) of the Act, or the EUA is revoked under Section 564(g) of the Act, or until such time that the approved configuration can no longer be maintained.



Warning

This respirator helps to provide protection against certain particulates.

Misuse of this product may result in serious illness or death.

For proper use, please see User Instructions, or contact PHG at <https://protectivehealthgear.com/> or call +1(800)225-1025.



MADE IN U.S.A.
of globally sourced
materials

Model 5160

Protective Health Gear LLC
35-57 Kentucky Ave.
Paterson NJ 07503
Tel: +1(800)225-1025
Fax: 973-689-6163
Sales@protectivehealthgear.com
www.protectivehealthgear.com



Particulate Respirator N95

Model
5160

PHG Protective
Health Gear

Particulate Respirator N95

Respirateur N95 contre les particules

Respirador contra partículas N95



50 Pieces Per Carton
Individually Packed



MADE IN U.S.A.
of globally sourced
materials



LATEX
FREE