

PARTICULATE RESPIRATOR N95 - MODEL 6150 / TC-84A-9525

Before occupational use of this respirator, a written respiratory protection program must be implemented, meeting all the local government requirements. In the United States, employers must comply with OSHA 29 CFR 1910.134, which includes medical evaluation, training, fit testing, and applicable OSHA substance-specific standards.

USER INSTRUCTIONS

- Respirator must be checked before use
- Respirator must be discarded after each use.
- Wash your hands before putting on a respirator.



HOW TO PUT ON YOUR N95 RESPIRATOR:

1. Unfold the flat fold respirator and place the respirator against your face with the nose clip across the bridge of your nose. Hold the respirator on your face and position the respirator under the chin.
2. Pull the bottom strap over your head, and position it around the neck, below the ear. Pull the top strap over your head, resting it high at the top of your head.
3. Place the fingertips of both hands at the top of the metal nosepiece.
4. Mold the nosepiece to the shape of the nose bridge by pushing inwards while moving your fingertips down both sides of the nosepiece.

CHECK THE SEAL – Place both hands completely over the respirator, being careful not to disturb the position, exhale sharply. If air leaks around your nose, adjust the nose metal described in STEP-4. If air leaks at the respirator edges, adjust the straps back along the sides of your head and perform the seal check again. If you cannot achieve a proper fit, see your supervisor. Do not enter an area requiring respirator use.

HOW TO REMOVE / DISPOSE OF A USED RESPIRATOR – Grasp the bottom and top straps of the respirator, and remove them without touching the surface of the respirator. Dispose of the used respirator in a waste container. Wash your hands after taking off your respirator.

Manufactured By: Protective Health Gear, 35-57 Kentucky Ave. Paterson NJ 07503

Tel: +1(800) 225-1025 Fax: 973-689-6163

Sales@protectivehealthgear.com www.protectivehealthgear.com



Warning

This respirator helps to provide protection against certain particulates.

Misuse of this product may result in serious illness or death.

For proper use, please see User Instructions, or contact PHG at

<https://protectivehealthgear.com/> or call +1-800-225-1025



Single-use



Keep Dry



-20°C - +30°C
68°F - 86°F